

Replacing Therapy Electrodes

When Should Therapy Electrodes Be Replaced?

corPatch easy therapy electrodes are a Single Patient Use product. They can deliver several shocks before they require changing.

Replace **corPatch easy** therapy electrodes for **adults** no later than:

- 24 hours or 50 shocks;
- 8 hours of continuous operation in pacer mode.

Replace **corPatch easy Paediatric** therapy electrodes no later than:

- 8 hours or 25 shocks;
- 1 hour of continuous operation in pacer mode.

Please adhere to the guidelines provided by your individual trust.

corpuls therapy electrodes



	Accessory Number	04324.3	05120.4	05120.5	05120.3
	Description	corPatch easy	corPatch easy Pro	corPatch easy pre-connected	corPatch easy paediatric extended
					
Device	corpuls aed	✓	✓	✓	✓
	corpuls 1	✓	✓	✓	✓
	corpuls 3	✓	✓	✓	✓
Patient	<20Kgs				✓
	>20 Kgs	✓	✓	✓	
Application	Defibrillation shocks	50	50	50	25
	Max. number of hours pacing	8	8	8	1
	Max. number of hours on skin	24	24	24	8
	HBO approved	✓	✓	✓	✓
Storage	Storage time	30 months	36 months	36 months	36 months
	Storage temperature	0°C – 50°C	0°C – 50°C	0°C – 50°C	0°C – 50°C