



Shock Test Change Notice



New Shock Test Protocol

There will be 2 shock test devices circulating.
Below are the differences in protocol for the **historical Test Box** and the **new Test Load**.

Test Box



- ⇒ Connect the **Test Box** to the therapy socket
- ⇒ Press the [Yes] softkey to confirm the message
“Mark mission as test mission?”
- ⇒ Select Manual mode
- ⇒ Select an energy level of **200 joules**
- ⇒ Charge
- ⇒ Deliver the shock



Check the shock protocol printout for the message
“Test Successful” OR “Test Failed”

Test Load



- ⇒ Connect the **Test Load** to the therapy socket
- ⇒ Press the [Yes] softkey to confirm the message
“Mark mission as test mission?”
- ⇒ Select Manual mode
- ⇒ Select an energy level of **50 joules**
- ⇒ Charge
- ⇒ Deliver the shock



Check the shock protocol printout for the message
“Test Successful” OR “Test Failed”



Ortus Academy

Ortus Academy is a trading name of Ortus Technology Limited.
Company Registration 07554900

ORTUS
ACADEMY

Academy_Test Load_Change Notice_SWV4.3_A048_v1

This document has been developed to provide users with an overview for safe device operation in accordance with its function and intended use and does not replace the user manual. All persons dealing with use, maintenance and troubleshooting must read and implement the user manual. Illustrations may not reflect that displayed on your device. Subject to technical modifications, mistakes, and printing errors.